

The logo for Zirlux FC features the word "Zirlux" in a blue sans-serif font, with a blue and yellow circular graphic element behind the "i". The letters "FC" are in a yellow sans-serif font. Below the main text, the words "FULL CONTOUR ZIRCONIA" are written in a smaller, white, all-caps sans-serif font.

ZirluxTM FC
FULL CONTOUR ZIRCONIA

**Strength.
Made Perfect by Beauty**

Why choose Zirlux FC full contour zirconia?

Regardless of the clinical situation, today's patients demand an esthetic restorative outcome that matches or looks better than their natural teeth. Traditionally, this demand has been met by the use of all-ceramic or layered restorations. However, many patients present with signs of bruxism or exhibit worn dentition that does not allow for the use of traditional ceramic restorative options.

In the past, the clinical and laboratory solution for the most challenging of cases was full metal or metal occlusal restorations. Today's solution is Zirlux FC full contour zirconia.

Pre-shaded zirconia

- 5 unique pre-shaded zirconia shades
- Minimize inventory – use 5 base shades to produce all 16 A-D plus bleach shades
- Monolithic strength - as little as 0.5 mm of reduction required
- Indicated for conventional cementation





Stain & Glaze technique

- Increased esthetic potential using familiar stain, modifier, and glaze pastes
- Consistent results – what you see is what you get
- Save time – eliminates the need for dipping, coloring liquid application, and drying time
- Fire in a traditional porcelain furnace



Stain & glaze to finish



Strength. Made Perfect by Beauty

- Zirlux FC brings beautiful esthetic outcomes to the strength of full contour zirconia
- Eliminate the esthetic unpredictability that arises when using hazardous coloring liquids found in other full contour zirconia systems
- Higher translucency than framework zirconia



Finished restoration

Preparation Guidelines

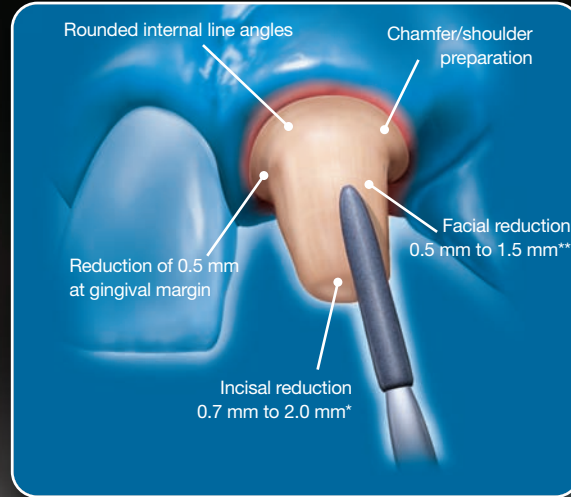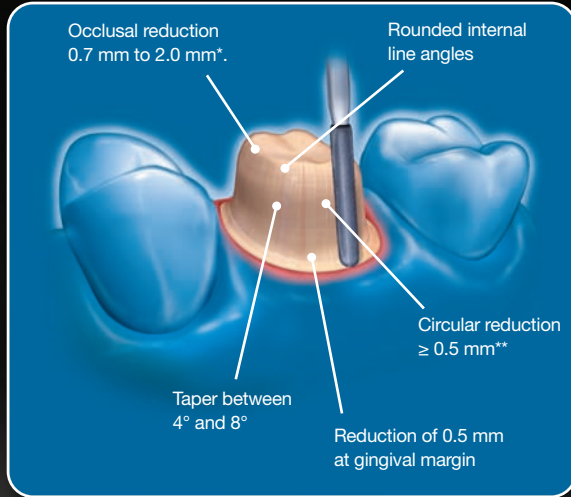


*1.0 mm minimum incisal/occlusal reduction for bridges

**0.7 mm minimum facial/circular reduction for bridges

©2011 Henry Schein, Inc.
Zirlux is a trademark of Zahn Dental.
Rev. 1 2011-02

For more information, contact Zahn Dental at 1-800-496-9500.

Manufactured For:

 **Ardent Dental, Inc.**
175 Pineview Dr.
Amherst, NY 14228
800-922-5520

 **AB Ardent**
8 Generatorgatan
SE-195 Arlandastad
SWEDEN

  
For Dental Use/
Made in USA